F

COMPOSITION

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Water-soluble lubricant, E.D.T.A., carbamide peroxide.

PROPERTIES

- lubricates the root-canal and facilitates endodontic instrument penetration,
- makes shaping more efficient,
- due to its effervescent properties, makes cleaning of the root-canal easier.

INDICATION

Facilitates root-canal preparation.

INSTRUCTIONS FOR USE

CANAL PLUS should be used repeating the following sequence:

- coat the instrument to be used with the gel for lubrication,
- shape to the desired number,
- irrigate with a sodium hypochlorite solution (2%-5%).

OPERATIVE TECHNIQUE

1. Pulp chamber

- Isolate the tooth to be treated, using a rubber dam;
- Open up the pulp chamber in the usual manner;
- *If using a syringe:* Fit the applicator tip to the CANAL PLUS syringe; *If using a single-dose applicator:* Sever the thin stem of the applicator with your fingers or using a cutting instrument;
- Place CANAL PLUS;
- Irrigate with a sodium hypochlorite solution to remove pulp debris and clear the entrance to the root-canal(s);
- Remove the liquid from the chamber by aspiration.

2. Root-canal

The way CANAL PLUS is used will vary, depending on the anatomical features of the root-canal(s) to be prepared. Incorrect direction of reaming should be avoided in any instance. For preparation of wide and relatively straight root-canals, CANAL PLUS can be used starting with the first instrument number. For narrow, calcified and/or irregular shaped canals it may be necessary to assess the shape and working length and to start reaming up to a rather high number before applying CANAL PLUS. The instruments should be pre-curved and adjusted (stopped) to facilitate the work.

Then proceed by coating the instruments with CANAL PLUS, working in the usual way ; then rinse with hypochlorite, dry and fill the canal as normal.

Notice

Avoid using CANAL PLUS right up to the last numbered reamer. Although it is highly water soluble, always make sure that no trace of CANAL PLUS remains in the canal.

CANAL PLUS should not be left in the tooth between visits.

SPECIAL WARNINGS AND PRECAUTIONS FOR USE

C- Corrosive (contains carbamide peroxide).

Causes burns.

In case of contact with eyes, rinse immediately with plenty of water and seek medical advice.

Wear suitable protective clothing, gloves and eye/face protection.

In case of accident or if you feel unwell, seek medical advice immediately (show the label where possible).

STORAGE

Keep in a cool place (2°C to 8°C).

PRESENTATIONS

5 g syringe + 15 applicator tips.

For professional dental use only

Distributor: SEPTODONT Ltd.

Unit R & S - Orchard Business Centre - St Barnabas Close ALLINGTON, Maidstone - KENT ME16 0JZ - UK

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