FR

COMPOSITION

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EN

COMPOSITION

Per sponge:
Native, non-denaturated, freeze-dried collagen
of bovine origin..... 0.027 g

TSE/BSE

The risk assessment for virus transmission and animal transmissible spongiform encephalopathies has been performed in compliance with the provisions of directive 2003/32/EC.

INDICATION

Local hemostasis in dentistry.

CONTRAINDICATIONS

Patients with a genetic predisposition to allergy or a known history of allergy to collagen of bovine origin.

DIRECTIONS FOR USE

Before placing the hemostatic sponge, re-examine the socket thoroughly to remove all granulation tissue and bone debris. Apply a dry sponge with or without pressure depending on the amount of bleeding, making sure that the socket does not dry out. If necessary, the sponge can be held in place with a cross stitch over the socket.

The sponge will be completely and naturally resorbed.

SPECIAL WARNINGS AND PRECAUTIONS FOR USE

- Do not use in infected areas.
- Do not moisten the sponge.
- Any non-used piece of sponge should be disposed of since it is no longer sterile.
- It is recommended:
 - to use dry, smooth instruments,
 - to apply small pieces of sponge, just big enough to fill the socket.

Check package integrity before use.

Do not use if package is open or damaged before first use.

Do not resterilize.

STORAGE

So as to keep the sponges sterile and prevent them from regaining moisture, do not remove the heat-sealed film or take the sponges out of their bags until use. Keep away from sources of heat.

PRESENTATION

Box containing 24 single use sponges.

Sponges radio-sterilised by gamma-ray irradiation.

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