(FR)

COMPOSITION

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(EN)

COMPOSITION

Powder composed of tricalcium silicate, zirconium oxide and povidone. Aqueous solution of calcium chloride and polycarboxylate.

INTENDED USE

Permanent root canal sealer

CLINICAL BENEFIT

Preservation of the tooth functionality

INTENDED POPULATION

Permanent and mature teeth

INDICATIONS FOR USE

Permanent root canal filling in combination with gutta-percha points in case of inflamed or necrotic pulp.

Permanent root canal filling in combination with gutta-percha points following a retreatment procedure.

BioRoot™ RCS is suitable for use in single cone technique or cold lateral condensation.

CONTRA INDICATION

Hypersensitivity to one of components
Immature or temporary teeth

PERFORMANCE CHARACTERISTICS

Technical performance

Seal hermetically the root canal.

Technical specifications

Setting time from 45 minutes to 4 hours allowing a sufficient working time.

Radiopaque, non-staining, dimensionally stable, non-soluble in fluids, adhesion to mineralized tissues and gutta-percha.

Properties

- BioRoot™ RCS is highly biocompatible and reduces the risk of adverse tissue reaction.
- BioRoot™ RCS crystallization creates a tight seal within the dentin tubules.
- BioRoot™ RCS fosters mineralization of the dentin structure and facilitates the physiologic bone process. BioRoot™ RCS therefore creates a favorable environment for periapical healing.
- BioRoot™ RCS limits bacterial propagation.
- BioRoot™ RCS can be easily removed from the root canal if a retreatment is required.

CLINICAL PERFORMANCE

Permanent root canal obturation of mature teeth.

EXPECTED LIFETIME

At least 5 years

MIXING INSTRUCTIONS

- Collect the powder with the spoon supplied in the box. Put a level spoonful of the powder on the mixing pad.
- Detach a mixing solution single dose container. Twist cap to open. Pour 5 drops from the single dose container on the mixing pad.
- Prepare the root canal sealer by progressively adding powder to the liquid. Mix until obtaining a smooth paste (about 60 seconds). The mix must be made extemporaneously.
- Immediately rinse and clean the instruments to remove any residual material.

PLACEMENT INTO THE ROOT CANAL

- Perform the root canal cleaning and shaping procedure using standard endodontic procedures.
- Select standardized gutta-percha master cone and check that it fits snugly at the working length.
- Enjuague con una solución salina estéril para eliminar cualquier residuo de solución irrigante y seque sin deshidratar totalmente el conducto radicular.
- Prepare BioRoot™ RCS according to the above mentioned mixing instructions.
- Apply a coating of BioRoot™ RCS onto the canal walls using either a paper point or the gutta-percha point.
- Complete the obturation by inserting the gutta-percha master cone previously coated with BioRoot™ RCS (single cone technique) or several coated gutta-percha points (lateral condensation technique).
- Assess the quality of the root canal filling with a radiography.
- Use a hot instrument to sever the protruding gutta-percha.
- Perform the temporary restoration using any temporary filling material.
- If a post placement is required, it should be performed after complete hardening of BioRoot™ RCS, i.e. between 1 week and 1 month.

RETRACEMENT PROCEDURE

Remove the existing sealer (BioRoot™ RCS or another sealer) using conventional removal techniques. Then follow instructions given in the sections "Mixing instructions" and "Placement into the root canal".

WARNINGS

- Do not use the mixed paste on several patients to avoid cross contamination.
- The working length in the root canal should not exceed 2 mm from the radiological apex to avoid apical extrusion.
- The device may cause eye irritation and skin burn. Do not manipulate the device without appropriate personal protective equipment.
- Use a rubber dam to ensure that the cement is not in contact with the mucosa or swallowed.
- Use the spoon provided with BioRoot™ RCS. Otherwise, technical specifications are not guaranteed and may lead to the failure of the endodontic procedure.
- Do not use BioRoot™ RCS powder with another liquid than BioRoot™ RCS liquid. Otherwise, technical specifications are not guaranteed and may lead to the failure of the endodontic procedure.
- Use the recommended ratio of liquid/powder. Otherwise, technical specifications are not guaranteed and may lead to the failure of the endodontic procedure.
- Do not use a warm technique obturation. Otherwise, technical specifications are not guaranteed and may lead to the failure of the endodontic procedure.
- Using BioRoot™ RCS without gutta-percha points jeopardizes endodontic and retreatment procedures.

DISPOSAL

No specific recommendations

STORAGE

Do not store above 30 °C; protect from moisture.

Use the powder within 6 months after first opening.

Liquid in single-dose containers must be used immediately after opening. Any remaining material shall be discarded.

Use the 5 liquid single-dose containers within 3 months after first opening of the foil.

PRESENTATION

Assembly kit for 35 applications, containing:

- 35 liquid single-dose containers
- 1 fracoo of 15 g powder
- 1 spoon

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