(FR)

**UTILISATION**

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**PROPERTIES**

Alveogyl is a paste for alveolar surgical dressing after tooth extractions, with Penghawar fibers. The paste promotes haemostasis by compression and protects from superinfection (alveolar osteitis) with its barrier effect.

Introduced into the socket, Alveogyl easily adheres to the alveolus due to its fibrous consistency. Assisted by the patient's tongue movements, it gradually self eliminates without any intervention by the practitioner as the normal healing of the wound takes place.

Alveogyl is very easy to apply and requires no special attention from the practitioner other than observation of the healing process.

**INDICATIONS**

Alveogyl is a paste used as dressing in case of dry socket or post-extraction dressing following a difficult or traumatic extraction in patients with history of dry sockets.

**CONTRA-INDICATIONS**

Alveogyl is not to be used on patients with history of hypersensitivity to one of the components and nor on post extraction of deciduous teeth.

**INSTRUCTIONS FOR USE**

Take a small pellet (about 0.20 g) of paste and place it gently into the prepared dental socket. Do not suture.

**WARNINGS AND PRECAUTIONS OF USE**
Patients should not wash their mouth vigorously within the 24 hours following the avulsion. Do not treat more than 4 teeth in one session.

Warning
Contains eugenol and mint flavour ((S)-p-mentha-1,8-diene)
Causes serious eye irritation.
May cause an allergic skin reaction.
Harmful to aquatic life with long lasting effects. Refer to safety data sheet (available upon request).

Replace the cap immediately after use.

**STORAGE**

Alveogyl is stable until the expiration date indicated on the box label, when stored at 5-30 °C.

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